# TANITA

Monitoring Your Health

## **Tanita Corporation**

14-2, 1-Chrome, Maeno-Cho  
Itabashi-Ku, Tokyo, Japan 174-8630  
Tel: +81(0)-3-3968-2123  
Fax: +81(0)-3-3967-3766  
[www.tanita.co.jp](http://www.tanita.co.jp)  
ISO 9001 Certified

## **Tanita Corporation of America**

2625 South Clearbrook Drive  
Arlington Heights, IL 60005 USA  
Tel: +1-847-640-9241  
Fax: +1-847-640-9261  
[www.tanita.com](http://www.tanita.com)  
(800) Tanita-8 (USA & Canada Only)

## **Tanita France S.A.**

Villa Labrouste  
68 Boulevard Bourdon  
92200 Neuilly-Sur-Seine France  
Tel: +33 (0)-1-55-24-99-99  
Fax: +33 (0)-1-55-24-98-68  
[www.tanita.fr](http://www.tanita.fr)

## **Tanita Health Equipment H.K. Ltd.**

Unit 301-303 3/F Wing On Plaza  
62 Mody Road, Tsimshatsui East  
Kowloon, Hong Kong  
Tel: +852 2838 7111  
Fax: +852 2838 8667

## **Tanita Europe GmbH**

Dresdener Strasse 25  
D-71065 Sindelfingen  
Germany  
Tel: +49 (0)-7031-6189-6  
Fax: +49 (0)-7031-6189-71  
[www.tanita.de](http://www.tanita.de)

## **Tanita UK Ltd.**

The Barn, Philpots Close, Yiewsley,  
Middlesex, UB7 7RY UK  
Tel: +44 (0) 1895 438577  
Fax: +44 (0) 1895 438511  
[www.tanita.co.uk](http://www.tanita.co.uk)

# TANITA

Monitoring Your Health


## Professional Raised Dial

# OPERATING INSTRUCTIONS

## Professional Raised Rotating Indicator Scale – HA-502

- ① Measuring Platform
- ② Dial Display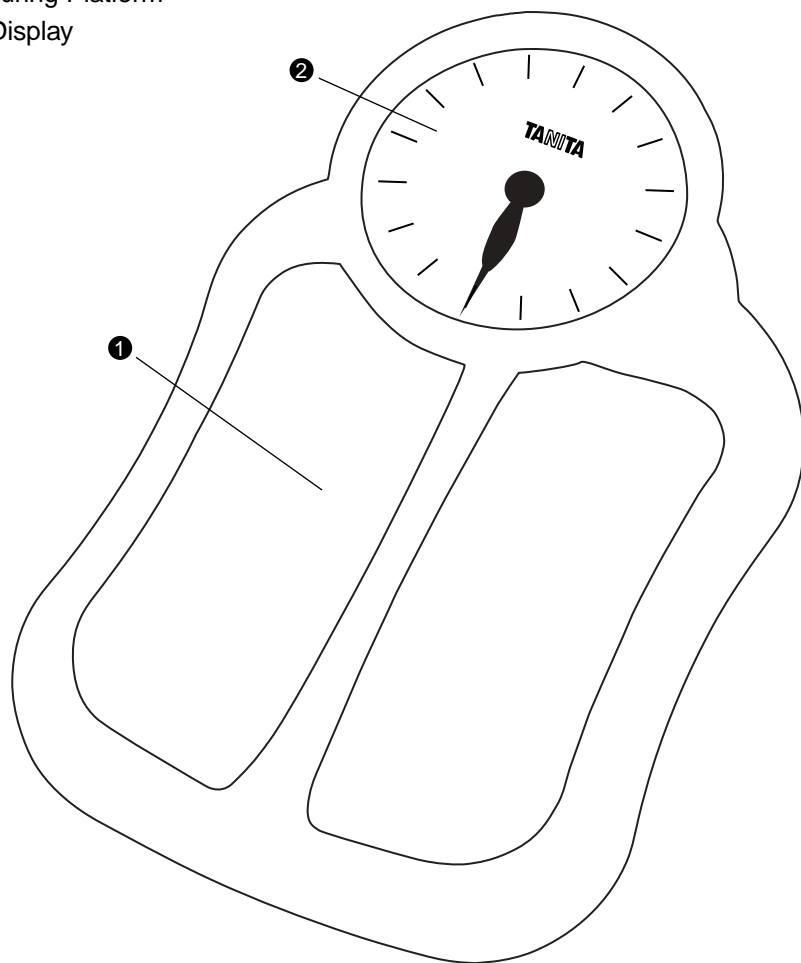

### Specifications

Maximum Capacity: 400 lb (180 kg)

Graduation: 1 lb (.5 kg)

### UK Specifications

Maximum Capacity: 23 st/lb (150 kg)

Graduation: 1 st/lb (1 kg)

### Features

- 7" raised white pearl dial
- Accuracy and reliability.
- Fits any décor.
- Durable construction, non-skid surface.

### Placing Scale

The scale should be used on a hard even surface for accurate weighing. Not recommended for use on carpeted surfaces. Placing scale on a very soft, deep pile rug may affect weighing accuracy.

### Instructions for Use

Before using scale, allow it to reach room temperature; then put scale through several weighing cycles consisting of:

Step on scale and allow dial to come to rest.

Step off scale and allow dial to come to rest.

After weighing cycles are completed, adjust dial to zero, if necessary (adjustment knob under edge of platform), and repeat weighing cycles until zero reading is stabilized.

If scale is relocated, readjustment of zero may be required. Repeat above procedure.

When weighing, place feet on platform so weight is centralized.

Place on hard, level floor surface.

### Care and Maintenance

An occasional wiping with a soft cloth and mild soap will keep your scale attractive. Do not use abrasive cleaners. Your scale does not require oiling or lubrication of any kind.

### After Sales Service

Please contact the Tanita office nearest you regarding operation or trouble, request for repair, and/or placement of an order for parts for products.

***Limited 10 Year Warranty with valid proof of purchase.***

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