# TANITA

Monitoring Your Health

## **Tanita Corporation**

14-2, 1-Chrome, Maeno-Cho  
Itabashi-Ku, Tokyo, Japan 174-8630  
Tel: +81(0)-3-3968-2123  
Fax: +81(0)-3-3967-3766  
[www.tanita.co.jp](http://www.tanita.co.jp)  
ISO 9001 Certified

## **Tanita Corporation of America**

2625 South Clearbrook Drive  
Arlington Heights, IL 60005 USA  
Tel: +1-847-640-9241  
Fax: +1-847-640-9261  
[www.tanita.com](http://www.tanita.com)  
(800) Tanita-8 (USA & Canada Only)

## **Tanita France S.A.**

Villa Labrouste  
68 Boulevard Bourdon  
92200 Neuilly-Sur-Seine France  
Tel: +33 (0)-1-55-24-99-99  
Fax: +33 (0)-1-55-24-98-68  
[www.tanita.fr](http://www.tanita.fr)

## **Tanita Health Equipment H.K. Ltd.**

Unit 301-303 3/F Wing On Plaza  
62 Mody Road, Tsimshatsui East  
Kowloon, Hong Kong  
Tel: +852 2838 7111  
Fax: +852 2838 8667

## **Tanita Europe GmbH**

Dresdener Strasse 25  
D-71065 Sindelfingen  
Germany  
Tel: +49 (0)-7031-6189-6  
Fax: +49 (0)-7031-6189-71  
[www.tanita.de](http://www.tanita.de)

## **Tanita UK Ltd.**

The Barn, Philpots Close, Yiewsley,  
Middlesex, UB7 7RY UK  
Tel: +44 (0) 1895 438577  
Fax: +44 (0) 1895 438511  
[www.tanita.co.uk](http://www.tanita.co.uk)

# TANITA

Monitoring Your Health


# Digital Lithium

# OPERATING INSTRUCTIONS

## Digital Lithium Scale – HD-372

- 1 Measuring Platform
- 2 LCD Display
- 3 Battery Compartment on Bottom of Scale
- 4 Weight Mode Switch on Bottom or Side of Scale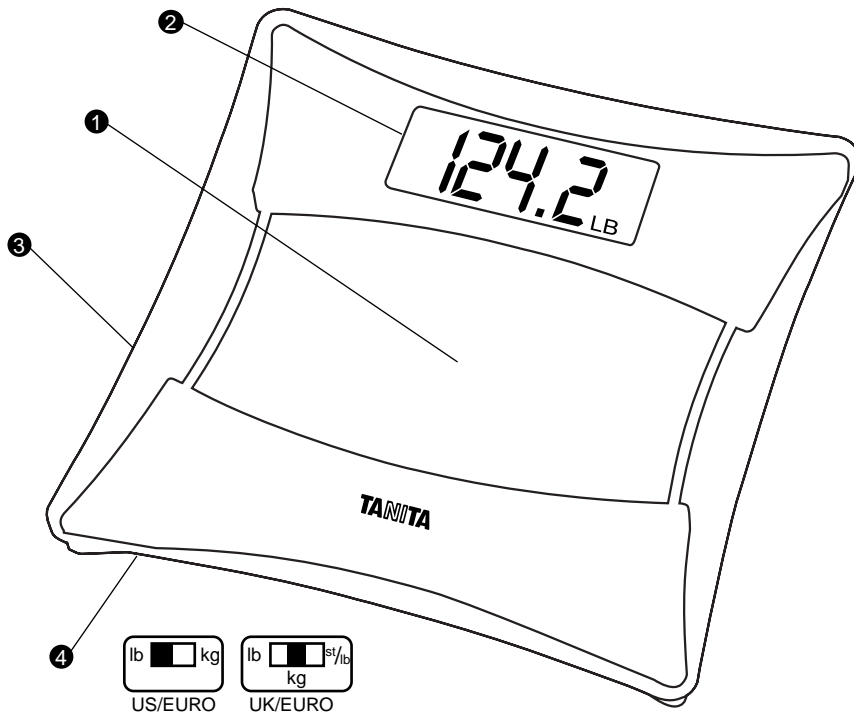

## Specifications

Maximum Capacity: 400 lb (180 kg)  
Graduation: 0.2 lb (0.1 kg)  
Power Supply: 1 x 3 volt CR2032 Replaceable Lithium Battery (Included)  
1 x 9 volt Replaceable Battery (included)

## UK Specifications

Maximum Capacity: 28 st/lb (180 kg)  
Graduation: 1/4 st/lb (0.1 kg)  
Power Supply: 1 x 3 volt CR2032 Replaceable Lithium Battery (Included)  
1 x 9 volt Replaceable Battery (included)

## Features

- Digital accuracy and reliability.
- Elegant, contemporary styling.
- Durable construction, non-skid surface.
- “L” message appears when battery needs replacing.
- Back light display

## Before Use

Select your preferred weight mode using the Weight Mode Switch

## Placing Scale

The scale should be used on a hard even surface for accurate weighing. Not recommended for use on carpeted surfaces. Placing scale on a very soft, deep pile rug may affect weighing accuracy.

## Instructions for Use

Gently tap the center of the scale with your foot. When “0.0” shows on the display, step on the scale (Fig. 1).

The display shows your body weight (Fig. 2).

Step off the scale.

The weight remains about 5 seconds. The unit then turns off.

Note: “Err” will appear on the display when the scale’s weight capacity is exceeded.

## Calibration

To recalibrate the scale, gently tap the center of the scale with your foot. Do not place your hand on the scale while calibrating (see Fig. 2); if you do, “-0.2” or “-0.4” may be displayed.



## Care and Maintenance

- Place the scale in an area free from heating equipment, high humidity, or extreme temperature changes.
- Never submerge the scale in water or spill chemical liquids on it. To clean, use a dampened cloth and mild detergent.
- Avoid excessive impact or vibration to the unit.

## After Sales Service

Please contact the Tanita office nearest you regarding operation or trouble, request for repair, and/or placement of an order for parts for products.

**Limited 10 Year Warranty with valid proof of purchase.**

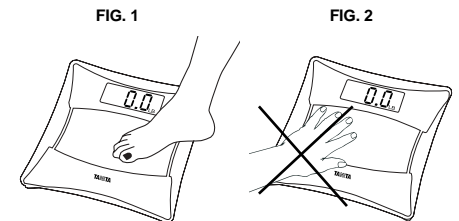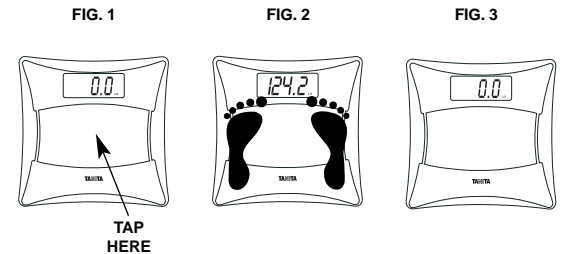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